



YOUR GUIDE

# WHICH RAMP SUITS YOU?



Yes, you can.®

# INVACARE RAMPS EVALUATION

We want to help you decide which INVACARE RAMP is right for you. Please see some questions below that will help you make the right decision.



Personal Model



Multi Purpose Model



EBL Model



Senior Model



## 1. Where are you using the Ramp?

- ☐ Used for easier access in and around the home or workplace, ideal for climbing kerbs and similar obstacles (select Personal)
- ☐ Used for climbing larger steps and higher kerbs than the Personal model Great for commercial or travelling situations (select Multipurpose)
- ☐ Used for where the doorway is between 75cm and 82cm, and is a great solution for loading unoccupied equipment into the rear of a vehicle (select Senior)
- ☐ Used for access to shops, offices and government buildings, and where unassisted on long ramps or where there are any risks involved, ie would you feel more secure with edge barriers on your ramp (select an EBL)



## 2. Measure and calculate total step/kerb height you will be needing

notes:



## 3. Refer to our Ramp Calculator and determine if there is an appropriate Ramp length at 4 degrees (AS/NZ standards)

- ☐ Yes ☐ No (move to Q4)



## 4. You need to complete a risk assessment to see if an increased angle gradient is suitable

- Ensure there is a suitably trained person to assist in the risk assessment
- Consider if the Ramp will be used with Power or Manual equipment
- Consider if the Ramp will be used with Manual assisted or unassisted equipment
- If a new selection is determined as suitable, move to Q5



## 5. Check that the ramp, once folded, fits into your transport requirements

- ☐ Yes ☐ No (If no select new ramp size)




## 6. Will you occasionally travel to locations you're unfamiliar with? Consider the multi purpose model which comes with a bag & weighs only 4kg (this may be an additional Ramp requirement)

notes:



## 7. If you need to move the ramp regularly, then consider the physical abilities of the user

# RAMP CALCULATOR

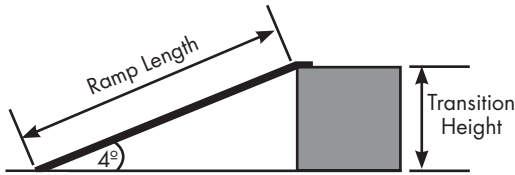

Ramp Length		Transition Height			
<b>Personal</b>	<b>70cm</b>	<b>5cm</b>	<b>9cm</b>	<b>14cm</b>	<b>18cm</b>
<b>Multipurpose</b>	<b>87.5cm</b>	<b>6cm</b>	<b>12cm</b>	<b>18cm</b>	<b>23cm</b>
	<b>100cm</b>	<b>7cm</b>	<b>14cm</b>	<b>21cm</b>	<b>26cm</b>
<b>Senior</b>	<b>EBL</b>	<b>90cm</b>	<b>6cm</b>	<b>13cm</b>	<b>19cm</b>
	<b>120cm</b>	<b>8cm</b>	<b>17cm</b>	<b>25cm</b>	<b>31cm</b>
	<b>135cm</b>	<b>9cm</b>	<b>19cm</b>	<b>28cm</b>	<b>35cm</b>
	<b>150cm</b>	<b>10cm</b>	<b>21cm</b>	<b>31cm</b>	<b>39cm</b>
	<b>165cm</b>	<b>12cm</b>	<b>23cm</b>	<b>34cm</b>	<b>43cm</b>
	<b>200cm</b>	<b>14cm</b>	<b>28cm</b>	<b>42cm</b>	<b>52cm</b>
	<b>250cm</b>	<b>17cm</b>	<b>35cm</b>	<b>52cm</b>	<b>65cm</b>
	<b>300cm</b>	<b>21cm</b>	<b>42cm</b>	<b>62cm</b>	<b>78cm</b>
	<b>350cm</b>	<b>24cm</b>	<b>49cm</b>	<b>73cm</b>	<b>91cm</b>
	<b>185cm</b>	<b>13cm</b>	<b>26cm</b>	<b>38cm</b>	<b>48cm</b>

Invacare recommends risk assessment is completed for use over 4 degrees

AS/NZ standards only apply to 4 degrees

\* The Ramp Calculator is indicative only to assist your judgement

## THANK YOU

for taking the time to complete our ramps evaluation. This will help us to ensure that we provide you with the **INVACARE RAMP** best suited to your needs. To find out more about Invacare products, speak with your retailer,

**CALL US ON 1800 460 460** or go to **www.invacare.com.au**