

User Manual

Ellipse Lite 2-way Rollator



Thank you for purchasing the Peak-Care Ellipse Lite 2-way Rollator. It can be configured as either a walking aid or a light weight foldable wheelchair.

The Walking aid is suitable for a single user with limited mobility. The Ellipse 2-way Lite is ideal for indoor and outdoor use.

Kindly take time to go through this manual which gives you more details about your Ellipse Lite 2-way Rollator. Kindly keep this booklet handy and use it as and when required.

The Ellipse Lite 2-way Rollator is warranted to be free of defects in material and workmanship for twelve months from the date of purchase

This device was built to exacting standards and carefully inspected prior to packing and shipping. This warranty is an expression of our confidence in the materials, our workmanship and an assurance of the same to our customers.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the unit.

This warranty does not cover device failure due to owner's misuse or negligence or normal wear and tear. This warranty does not extend to nondurable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

This equipment does not contain any user serviceable parts. Any damaged components of the rollator should only be replaced with genuine, original parts available from Peak Care Equipment.

If you have any queries, please contact your place of purchase.

Kindly attach your receipt to this booklet

PARTS LIST



- | | |
|--|--------------------------|
| 1. Handgrip | 8. Front wheel |
| 2. Brake lever | 9. Seat |
| 3. Interlock button for handgrips | 10. Bag |
| 4. Release for adjusting handgrip height | 11. Footrest support arm |
| 5. Wheelchair mode arm rest | 12. Footrest |
| 6. Rear wheel | 13. Backstrap |
| 7. Brake adjust screw | |

INTRODUCTION

The 2-way rollator is suitable for a single user with limited mobility. It is ideal for both indoor and outdoor use.

THE ELLIPSE LITE 2-WAY ROLLATOR PACK INCLUDES

| | | | |
|---|---|--|---|
| <p>1 Ellipse Lite Rollator</p>  | <p>2 Bag</p>  | <p>3 Back Strap</p> <p>1027704</p>  | <p>4 User Manual</p>  |
|---|---|--|---|


UNFOLDING ELLIPSE Lite 2-way Rollator

1. Push the side frames outwards.
2. Push downwards on the seat rails to engage the locking mechanism.




HEIGHT ADJUSTMENT

1. Release the red clamp by pulling out from the frame.
2. Press the grey release button located below the handgrips as shown and at the same time either push the handle down (lowering handle height) or pull the handle up (raising handle height), releasing the push button will automatically lock the handle at the desired height.
3. Close the red clamp.

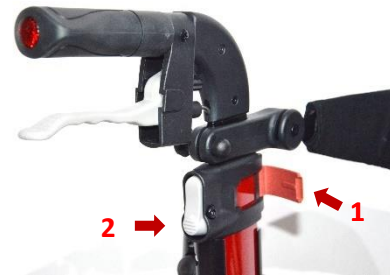

ATTACHING THE CARRY BAG

1. Fold the Ellipse Lite 2-way Rollator for ease of access.
2. Locate the bag hooks under the seat frame.
3. Slide the loops on the bag onto the hooks.

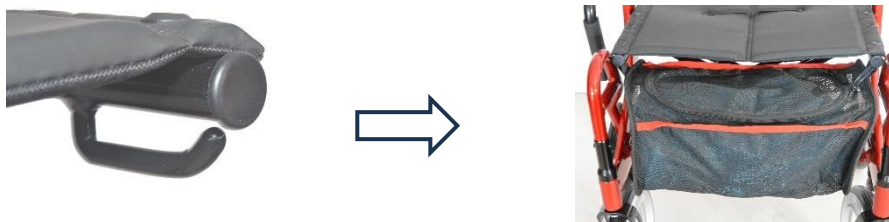

BRAKING & LOCKING THE BRAKES

Pull both brake levers upwards to apply the brakes while the Ellipse Lite 2-way Rollator is moving.

Always brake slowly and steadily.



Press both levers down to lock the brakes while the Ellipse 2-way Lite is stationary.

Pull up the brake levers to release the locked brakes.



NOTE: The effectiveness of the brakes can be reduced after a period because of abrasion on the wheels. Brakes should be tested regularly, and adjusted to ensure they hold the rollator still when applied.

ADJUSTMENT OF BRAKES

To adjust the braking strength, adjust the brake by pressing the screw housing downwards and then left or right



SITTING WHEN USED AS A ROLLATOR

Position the Ellipse Lite 2-way Rollator on an even and solid surface.
Lock the brakes (parking position). The Ellipse Lite must stand and must not slip.

Sit straight up and do not lean backwards.




CONFIGURING AS A WHEELCHAIR

1. Lift up the Arm Rests until they lock into place.



2. Lift up and rotate the footrest arm into place.

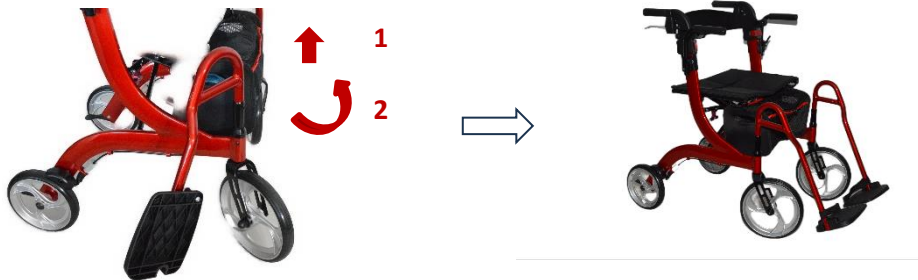

3. Fold the Back Strap to the rear of the frame.



FOLDING/ TRANSPORT/ STORAGE

1. Stow the armrest by pushing the lock release button
And push the rest down to its storage position.





2. Fold up the footrest, then lift and turn the footrest arm to return it to the folded position.



3. To fold, pull seat upwards by the handle of the seat plate



4. Once the Ellipse Lite 2-way Rollator is folded, the red locking clip will secure and stop the walker from opening whilst lifting in or out of a vehicle. See below:

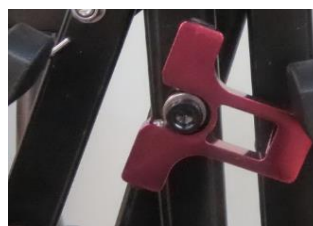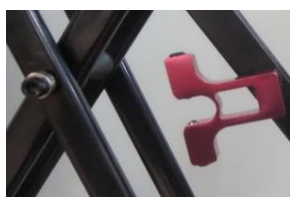

CLEANING AND MAINTENANCE

- Clean regularly by hand using a damp cloth.
- Do not use aggressive detergents or solvents.

SAFETY INSTRUCTIONS

IMPORTANT ➔ Do not propel yourself or have anyone push you whilst you are seated on the Ellipse 2-way when using it as a Rollator. In this mode it is a walking aid only and is not to be used as a transportation device or to be used to assist the user to rise from a sitting position.

IMPORTANT ➔ Do not attempt to stand while you are seated on the Ellipse 2-way when using it as a wheelchair as it could cause the wheelchair to tip forward. When dismounting while in wheelchair mode, first fold the footrest to reduce the risk of trip hazard and ensure both feet are firmly placed on the ground.

- Use only on even and solid surfaces.
- Maximum user weight: 130 Kg.
- Regularly check the screws and fasteners on the Ellipse 2-way Lite Rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5 Kg
- Do not expose the product to temperatures above 40 °C for an extended period of time, as this can cause the Ellipse 2-way Lite Rollator parts to overheat and burn the user.
- Make sure the Ellipse 2-way Lite Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position when used as either a rollator or wheelchair.
- Ensure the handgrips are adjusted to a comfortable height before use.
- Ensure that both knobs used to adjust the backstrap height are fully tightened and are at equal heights. Test the rollator for stability before applying full weight to the hand grips.
- Test the brake lever before operating the rollator. When wheels are locked, the lock bar on the rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use the seat when rollator is on an incline or uneven ground.
- Replace castors when tyres are significantly worn or damaged.

SPECIFICATIONS

| | |
|-------------------------------|---|
| Product code | 9346376000772 |
| Dimensions | Open - 850mm High x 610mm wide with 470mm wheelbase Folded – 850mm high x 270mm wide with 470mm wheelbase Seat – 425mm wide x 220mm deep Seat height - (from ground) – 510mm Handgrip height adjustment – 840mm to 965mm |
| Weight | 11.00 kg |
| Construction materials | Frame – Aluminium Carry bag – Nylon cloth and mesh Seat – Polyurethane cloth and foam |
| Max Carrying Capacity | 130kg |
| Max Carry Bag Capacity | 5.00kg |



Peak Healthcare Equipment Pty Ltd

Level 1, 149 Princes Highway

Unanderra NSW 2526

Australia

Tel: +61 2 4272 2688.

Fax: +61 2 4272 2699

www.peak-care.com.au

- This page intentionally left blank -

- This page intentionally left blank -

-